

HANDI-MOVE INTERNATIONAL PRESENTS

 **SUREHANDS**
LIFT & CARE®



Maintaining the Dignity and Freedom of those with Limited Mobility
Tailored, Reliable, Turn-Key Solutions



SUREHANDS® solutions

Making homes and workplaces work better	2
A philosophy of empowerment	4
Ready when you are	6
What users say	7
Tailored Solutions for any circumstance	10

Body Support Systems

The Body Support	16
Thigh supports	19
Slings	20
Spreader Bars	26

Lift & Care Systems

The 2600 series mobile lifts	29
Mobile lift model 2600/2610 (Victor)	30
Stand Assist model 2620	32
The 1600 series mobile lifts	34
2800 Series Ceiling Lift	38
2840 Series Portable Ceiling Lift	40
Ceiling Track Installations	42
Points / Turntable	44
Traverse Track Systems	45
The Freedom Bridge	47
The Wall Lift System	48
The Motorized Wheelchair-to-Water™ Pool Lift	50
The shower-bathing trolley	52

ACCESSORIES


Transport Trolley for the Portable Motor	54
Scales	55

SUREHANDS®

A powerful formula	56
--------------------------	----

SUREHANDS® from Handi-Move International

Making homes and workplaces work better

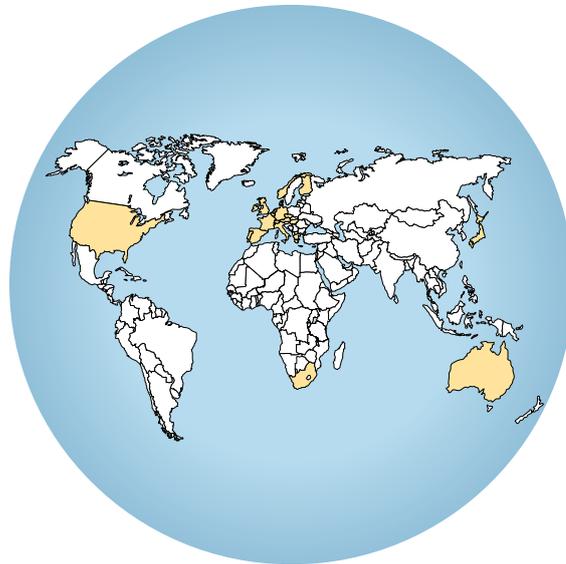

Global presence

Our systems and services are available in Europe, North America and Australia. Trained sales and service professionals are busy assisting and advising wheelchair users, their caregivers, doctors, therapists and other professionals on the solutions that will offer more freedom and quality of life for all. We've provided sound solutions to more than 50,000 people - all with various degrees of ability and each one with a unique set of goals.

Serving Private and Professional Markets

Our Lift and Care® Systems are popular with both private and professional consumers. Wherever there is a need for a practical and innovative approach to safe patient handling or self care, we have the answer. Accessing the bed, bath, or toilet are the immediate objectives, but changing clothes, catheterization, standing and ambulating, taking a swim or even mounting a horse are goals that can be met by many. Handi-Move's mission is to continue to innovate in the user's best interest.

Throughout the United States, SUREHANDS® representatives work closely with care providers in the private and professional marketplace. We perform site evaluations and patient assessments and then recommend appropriate systems to best meet the need and the budget. Our designs and drawings will insure that systems are discreetly incorporated into the home or professional environment.

In the user's best interest

We recognize the special needs of the population we serve. Our ability to solve our customers' problems is the key to our success. Each new request is a challenge to find a solution. The goal is to give every customer expert advice. Frequently our best solutions are the most economical. We train employees and representatives to offer responsible solutions consistent with our ethical philosophy.

A partner forever

A SUREHANDS® representative understands the important nature of his or her work and acts more as a guide and advisor than a sales person. A thorough explanation and evaluation covers the process through installation. After the sale, training is conducted by the representative. Proper usage and maintenance are reviewed. Our customers can count on unparalleled service response throughout the generous warranty period and beyond.



Over 40 years ago Handi-Move's founder, René Van Raemdonck, answered the call of a friend in a wheelchair. He designed a unique system of transfer for both self-lifting and attendant care. Further development by a university research group helped to patent and bring to the market the unique SUREHANDS® Body Support.

New innovations continue to address the needs of our users and caregivers. Our worldwide distribution makes it ever more possible for all people to have access to these innovations.



A philosophy of empowerment

In every phase of working with customers, SUREHANDS® strives to offer an outstanding service. Therefore, we have developed a company philosophy based on accessibility and clarity. From initial advice to the smallest tip on use, everything takes place in the same atmosphere of openness. No matter what the problem may be, we strive to have someone at your disposal to assist and advise. Friendly reception, sound purchasing advice, careful follow-up of orders, and professional service: these are the hallmarks of SUREHANDS®.

Partnering with architects and designers

Building living spaces for people with motor disabilities and their caregivers and is no easy matter. Architects and designers must take many special requirements into account. It is best to plan the lift system in the building concept phase. Our representatives are always ready to assist and advise designers. Habitat for Humanity recently took advantage of this free service.



Free evaluation

No matter how powerful a lift system might be, if it does not meet your needs it does not really help you. So, it is wise to first inform yourself thoroughly. Our SUREHANDS® advisors will gladly meet with you and provide you with advice that is based on deep practical knowledge and extensive experience. In this way, you are sure that you are purchasing a customized lift system that matches your needs and desires.

Free test

The SUREHANDS® representative is equipped with a mobile lift system, body support and slings so that you can try before you buy.

Crystal clear information

Detailed quotation

SUREHANDS® doesn't want you to be confronted with unexpected expenses after the purchase. Each quotation provides a clear picture of what the lift system will cost. The drawings and sketches can help you feel confident that your solution will be a success.

Comprehensive instruction guide

Each SUREHANDS® system is accompanied by a comprehensive instruction guide in which you will find the necessary explanation about the various components and how you can run the lift system. The guide gives you useful tips on maintenance and tells you where to turn if you should need assistance.

Step-by-step training

After installation, your representative will provide step-by-step training to insure a proper understanding of the usage and maintenance of your SUREHANDS® lift system.

Solid terms of warranty

You benefit from the best terms of warranty on all SUREHANDS® Lift & Care Systems and accessories. In the unlikely event of a problem with a part, it is quickly replaced during the warranty period - many times right on site.



Ready when you are

With SUREHANDS® Lift & Care Systems, collaboration with the customer continues even after the delivery. A growing number of local contractors are being trained in the proper installation method to provide consistent quality to our customers.

Qualified installers

Proper installation

A proper installation begins with an accurate scale drawing which reflects the desires of the prospective customer. From this drawing the proper quantity and style of track and accessories are selected so that the need for customization on site is limited. This also enables the installation to be completed in a minimum amount of time.

Tests after installation

Once the system is installed, it is tested for function and smooth operation. An experienced installer will be able to answer the customer's questions and provide helpful advice regarding usage and maintenance.

Customer support

Rest assured that anytime after your installation, you will be able to reach a qualified SUREHANDS® representative with your concerns. Sales representatives are supported by a national distribution center with customer support personnel who are also available to answer your questions. We are proud of our quick response time and ability to work on your concerns, either at our office or through an on site visit.

From inspiration to installation

SUREHANDS® offers you a comprehensive service from concept to realization. Our team of professionals can speak to any facet of the project and are available to assist with information required by agencies or funding sources. Our goal is to see the project through in close collaboration with you and according to plan.



What users say

I can count on SureHands to be my perfect “assistant” and get the job done right

I have been using SUREHANDS lifts on a daily basis since Junior High School and they have followed me from home, through college, then graduate school where I received my Ph.D in clinical psychology, and now at my place of work where I am a psychologist.

My professional image as a psychologist and runway model is very important, and no other lift system could complement this image better than SUREHANDS lifts. From getting ready in the morning to go to the clinic and treat my patients to getting glammed up at night to hit the Red Carpet, I can count on SUREHANDS to be my perfect “assistant” and get the job done right. The SUREHANDS lift is truly an amazing product from a reputable company that offers fantastic customer service! I would not be the successful, independent, glamorous woman that I am today without SUREHANDS lifts.

*Danielle Sheypuk, Ph.D. Psychologist and Model
Spinal Muscular Atrophy, Type2
Ms. Wheelchair New York 2012, USA*



What users say

“The SureHands® lift has not only provided safe and easy transfers but has promoted a sense of independence and confidence in our student.”

“Karl Davis is a 13 year old boy who attends seventh grade at Valley Central Middle School. He has a medical diagnosis of Duchenne’s Muscular Dystrophy. He negotiates the school environment using a manual wheelchair and with the assistance of a 1:1 Para-professional. Karl requires maximum assistance of two people to lift him in and out of his wheelchair when the lift is not used. He requires maximum assistance with activities of daily living. The SUREHANDS® mobile lift and care system is used to transfer him in and out of his wheelchair during therapy and for toileting. The lift has made it possible for his Para-professional to



assist Karl in the bathroom and to transfer him to the floor for rest periods without the assistance of another person. The lift promotes ease and safety for both Karl and his Para-professional. Karl uses the controller himself which gives him more independence as well as makes him feel more confident. Our representative lives in our area and has always been available to assist us in any way possible. We have been very pleased with the SUREHANDS® lift and care system.”

Patty Arnott

PT - Valley Central, Middle School, Montgomery, New York

“SureHands® has moved us into the 21st century with EASE”

“Our large (546 bed) residential, state facility has approximately 150 non-ambulatory, developmentally disabled individuals, many of whom are aging, medically fragile and with multiple handicaps. A large component of Physical Therapist’s time includes ongoing in-servicing on proper body mechanics and manual lifting techniques as well as instruction on mechanical lift systems.

The introduction of SUREHANDS® to our facility has markedly changed our service delivery system and has greatly enhanced services to our individuals. An immediate positive impact was realized as we obtained both the mobile units and the overhead ceiling track system for the bathrooms. SUREHANDS® staff provide excellent, thorough in-servicing for all staff and work closely with the Physical Therapy personnel in coordinating the initial and follow-up in-servicing. Direct care staff appreciate the ease with which the system can be operated and there is increased efficiency in the overall daily schedule.

The style of slings give options to the staff for the most comfortable and effective lift position from reclining to more upright sitting posture. Slings are easy to apply, do not require a manual lift to get into place, and provide comfort for our medically fragile individuals.

Safety is improved with the reduction of lifting injuries, and individuals feel safe and secure and relaxed during the lifting process. Staff can now provide multiple lifts required each day with greater efficiency and individuals can be transferred to and from beds, toilets, mat table, wheelchairs, mats on the floor, etc. with ease.



The overhead ceiling track system was designed to allow the transfer of the individual from wheelchair to toilet to bathing table and does not take up valuable floor space in the bathrooms. The combination of the ceiling track systems and the mobile units have changed the way we do business here. We continue to improve our facilities with the addition of newly renovated bathrooms, complete with the SUREHANDS® ceiling track system and high-low tub units. Increased safety, comfort, convenience and efficiency afforded our facility through the use of the SUREHANDS® system has significantly changed the lives of our staff and individuals.”

Sherry Rowe
PT Director in a large state facility, North Carolina

“You’ve given me my life back.”

“It’s so nice not to have nurses come in to take care of me anymore... all because of my SUREHANDS® lift system. And, yes, I’m still taking showers for as long as I want even though it’s been seven years since I purchased my lift. Recently I calculated the amount of money I have saved in nursing costs since I got my lift. It’s approximately \$250,000. Keep in mind this does not account for inflation; I’m sure they make far more now!!!

My lift was installed on the ceiling over my bed and runs into my bathroom so I can toilet and bathe independently. My SUREHANDS® sales representative has always responded really fast whenever I had a question or when I wanted my battery charger moved. He gets things done quickly and efficiently and has always given me peace of mind. I’m so grateful that I can still be home alone because of my lift.

Recently we moved to a warm and sunny climate and my lift moved with me. I could not live without it. I now also have a SUREHANDS® pool lift at my new home, and oh how I enjoy relaxing in the pool.

I feel like a bird that’s been let out of its cage.”

Sandy Holden
SCI C5/6 Quad, independent user



Tailored Solutions for any circumstance

The versatility of SUREHANDS® ensures that you will always have the correct solution for whatever circumstances you encounter.

Self transferring at home independent use

With the SUREHANDS® body support and ceiling system, some can transfer between bed, bath and toilet with little or no assistance.

Home care with an attendant

SUREHANDS® specializes in customized solutions for the home. The needs of the end user, the caregiver and the structure will be carefully considered by the representative to recommend the best SUREHANDS® lift suited to accomplish the goals.



A ceiling track system can connect you between your chair, bed, bath, and toilet to make easy efficient transfers. An authorized SUREHANDS® installer will make the necessary modifications to the home so the optimal track design can be provided.

If rooms are far apart, on different floors or even in different homes or facilities, the wall lift or portable ceiling lift can be ideal solutions. They are light weight and easy to move from one location to another. Versatile like a mobile lift, these could be perfect alternatives when the mobile lift is not a viable option.





A mobile lift can be considered for in-home use when the floors and doorways allow for maneuverability to access the areas of transfer. If space allows for a mobile lift, SUREHANDS® offers a variety of practical and safe alternatives.

Rely on SUREHANDS® to provide the very best lift solution that can be used independently or with an attendant.

Schools & Day Treatment Programs

Toileting & hygiene are the most common reasons for transferring students with disabilities in public and private school settings. The SUREHANDS® body support and hygiene sling are ideal for this purpose. Both allow the caregiver to have full access to the perineal area while the user is suspended in the lift.

SUREHANDS® has great experience providing solutions for the tight bathroom spaces often found in schools.

All SUREHANDS® systems will lower the user to the floor mats in the classroom or gymnasium.



Long Term Care

A variety of lift solutions help long term care facilities safely transfer patients while preventing injury to staff. Anywhere a transfer is needed SUREHANDS has a solution. Utilizing an overhead ceiling system in the bathing/shower area provides additional safety to the caregiver on wet floors, faster and smoother transfers to the resident and a lift system that is available when needed. To transfer to the toilet or other seat to seat transfers, a stand assist is compact and quick to use. Mobile lifters are versatile and can be moved to multiple locations where transfers are needed .

Hospital

The wall lift can help hospitals accomplish the goals of safe patient handling. Lifting onto specialized equipment such as X-Ray or operating tables, where traditional mobile lifters or ceiling track systems may not be feasible, the wall lift system emerges as an ideal solution. Its design facilitates smooth and efficient transfers, ensuring patients can be safely positioned for medical procedures with minimal strain on staff.





In patient rooms where flexibility and cost-effectiveness are paramount, the portable ceiling lift presents an ideal and economical option. Its lightweight construction and ease of use make it suitable for a variety of lifting tasks within the hospital setting, providing versatility and convenience where needed most.

Recreational

Swim therapy can be enjoyed in both private and professional environments with the use of either the SUREHANDS® ceiling mounted or deck mounted pool lifts. These are the only lifts that transfer a person directly from the wheelchair to the water. A ceiling system properly mounted over the pool, together with the SUREHANDS® body support, makes ambulation and other pool therapy possible.





Vacations are now easier knowing that SUREHANDS® systems are installed in a number of rooms at some of the most famous hotels in Las Vegas.

Horseback riding is a very popular therapy for many people with disabilities. Getting on and off the horse is now very easy with the SUREHANDS® body support and ceiling track system.



The Body Support

No shoulder strength needed

The SUREHANDS® body support lifts in the same way a mother lifts her child; her hands embrace, comfort, and provide a feeling of security. The body support goes on easily and provides that same feeling while leaving essential parts of the body accessible for care. The body support is applied easily and offers the user full functionality.

Self transfers

With a SUREHANDS ceiling motor, a person who has sufficient muscular tone in his hands will be able to transfer independently between bed, toilet, bath etc. using the SUREHANDS body support.

Ideal for the toilet

As the seat and back are completely free, the user is able to easily dress and undress.

Swimming Therapy

The Wheelchair-to-Water lift with a SUREHANDS body support takes the user directly into the water from the wheelchair.

Hippotherapy

The unique posture provided by the SUREHANDS body support makes transferring onto and off of horses very easy.



Uniquely designed

The body support works on a hinge system that is activated by the body weight of the user in the thigh supports thus causing the chest supports to automatically embrace and secure the upper body. It self-adjusts to an individual's body profile making it suitable for both children and adults.

The body support attaches in the same way as the spreader bar and therefore makes it very easy to change from the body support to slings if needed..



Hygienic

The body support is made from stainless steel and synthetic material. The body support is made from stainless steel and synthetic material which is easy to keep clean and disinfect as needed. All components are water tight.

Comfortable

The body support offers a high level of comfort and security to both the user and the caregiver. The curve-around cups embrace the torso at the same moment that the ergonomic thigh supports take weight of the lower extremities. The user's body weight assures the correct pressure so the optional cushioned covers are rarely needed.

Compatible with all SUREHANDS lifts





Thigh supports

The key to making the body support self adjust for you

Thigh supports are essential when lifting someone using a body support as they evenly distribute the body weight. They fit securely under the thigh and ensure that the body is perfectly supported when lifted in a sitting position. They can easily be fitted (and changed in either direction) to a person's thigh, either to the inside of the user's thighs or to the outside. They are available in different models and sizes, and with a smooth or slip-proof coating.

Thigh support fix

The patented thigh support fix™ supports a sustained good and comfortable position in the body support.

Under the pressure of the thighs of the user the self-locking feature holds the leg supports in its current position, once the tension is gone, it can easily be moved to another position.



The length of the thigh support bands are adjustable to your size and the markings allow you to easily verify that both thigh support bands have the same set length.

3 versions

Depending on the disability, it is important to choose the most suitable thigh supports: Standard Thigh Supports (suitable for most users), Contour Thigh Supports (intended for those with reduced sitting functionality) or Muscular Dystrophy Thigh Supports (in the event of strongly reduced muscular tone).



Slings

The right sling for every circumstance

Persons who are unable to control their head or who are otherwise contraindicated for use of the body support will benefit from the wide range of SUREHANDS® slings. The innovations and safety controls found in our other products are readily apparent in this quality range of slings.

Comfortable, strong and durable

SUREHANDS uses materials that are comfortable, strong and durable. Slings are available in mesh, solid fabric, parachute fabric or PVC. Most slings are machine washable and tumble dryer safe. They are available in various sizes and models. All SUREHANDS slings are latex-free.

For smooth transfers

Transfer smoothly to and from your bed, bath, toilet, wheelchair and shower thanks to the SUREHANDS slings. They can be used for transfers in lying or sitting positions, and are available with or without head support.



Professional advice

Our representatives will gladly assist you in choosing your sling. Most models and styles are available for demonstration and are available from stock.

Made to measure

SUREHANDS has years of experience and knowledge in made to measure slings specifically tailored to your needs.





Standard/contour sling

for universal use

The standard/contour sling is our most supportive sling. It fully supports the body and has a cut-out for toileting. And, patients can be easily positioned in or removed from the sling when they are safely placed into their wheelchair. The SUREHANDS® four point spreader bar creates extra room and allows the user to sit or lay more comfortably than from a straight two point spreader bar. The padded legs feel soft and offer ideal support at the same time. This padding also prevents the leg flaps from binding or bunching up. Positioning handles are sewn into the back of all our slings. The standard/contour sling is available with or without a crescent shaped head support. When more targeted head support is required, the six strap models are ideal. The standard/contour sling is available in solid fabric or mesh. A version without padding is ideal when the sling needs to remain underneath the user.



DayLite™ Sling

ideal for sensitive skin

Based on the standard contour sling, the DayLite™ sling is made of a parachute quality nylon that is very strong but with a smooth and soft texture to help prevent skin irritations. The exclusive thin fabric allows it to comfortably remain under the user as it actually wicks moisture away.



Hygiene sling

ideal for toileting

Sometimes referred to as a toileting sling, this model is especially designed to make toileting possible while in the security of the sling. It is so easy to put on that sometimes it can even be used independently. The large cut-out area behind the thighs and up the lower back allows access to the clothing so that the belt or waistband can be loosened and the user's pants can be removed while he or she is in the sling. A wide velcro belt is closed around the user's abdomen for extra safety and support. A version with head support is also available. This sling is often referred to as the "soft hands" because it is the next best thing in ease of use to the SUREHANDS® body support.



Thorax hygiene sling

The Thorax hygiene sling offers more support than a regular hygiene sling. The cut out area behind the thighs and up the lower back remains more free compared to a standard sling but less than a hygiene sling. The sling includes a self-adjusting system around the waist for even better support.



Bathing sling

pvc for quick drying in wet areas

The shape of the bathing sling is identical to that of the hygiene sling - only the material is different. The bathing sling is made of maintenance-friendly, latex-free, slip-proof, synthetic material that can be easily wiped dry. The bathing sling can be used dry as an alternative to the hygiene sling. The bottom and lower back remain free thanks to the large opening, ideal for intimate care.



Seat sling PVC

The SUREHANDS seat sling, made of synthetic latex-free material, is useful for people who can stand but lack stability and balance. It can be used in the home or institution. It can support the user in a 90 degree seated position with a firm seat and back support. It can also recline to other positions and is ideal for leaving underneath the user when lying in the tub. The perforated seat lets the water through. It can be attached to any of our systems. An optional security belt is available for those with poor balance in a seated position.



Pool lift chair

Transfer the swimmer to the pool with a sturdy and buoyant pool lift chair. This sturdy chair can be pre-positioned in the wheelchair, at a convenient location such as the changing room, making transfers more accessible. The swimmer sits in the chair and is brought to the pool lift. A caregiver can then attach the loops from the lift chair to the spreader bar and transfer the swimmer to the pool.

The chair design allows the swimmer to get out independently once the water reaches chest height. The density of the plastic and the presence of water drainage holes keep the chair partially buoyant, making it easy for the swimmer to get off and on.



Amputee sling

For people with an amputation above the knee

For people with a single amputation above the knee: a standard sling in mesh without head support with extended leg straps can be used.

For people with a double amputation above the knee: a rectangular sling without head support, with or without a toilet opening (available in different materials: mesh, solid fabric or parachute quality nylon fabric). The amputee sling is comfortable and safe and perfectly balances the body.



Walking harness

for gait training and ambulation

The ability to adjust this harness with numerous belts and buckles makes it an ideal tool for properly supporting the shoulders, stomach, chest and legs during gait training and ambulation. The safety strap further supports the user if fatigue is a problem during training.



Bariatric sling

SureHands also offers a suitable solution for the bariatric market.

STANDARD	Leg padding	Head-support	XS	S	M	L	XL
sling mesh fabric	-	-	-	3484S	3484M	3484L	-
sling mesh fabric	✓	-	3481XS	3481S	3481M	3481L	3481XL
sling mesh fabric 6 strap	-	✓	3437XS	3437S	3437M	3437L	3437XL
sling mesh fabric	✓	✓	3431XS	3431S	3431M	3431L	3431XL
sling solid fabric	-	-	-	3487S	3487M	3487L	-
sling solid fabric	✓	✓	3424XS	3424S	3424M	3424L	3424XL
DayLite Sling parachute nylon*	✓	✓	3445XS	3445S	3445M	3445L	3445XL
comfort sling	✓	✓	-	3451S	3451M	3451L	-

* version bariatric artn°.3530

HYGIENE	Leg padding	Head-support	XS	S	M	L	XL
2 piece sling	✓	-	-	-	3489M	-	-
hygiene sling	✓	-	3461XS	3461S	3461M	3461L	3461XL
hygiene sling 6 strap	✓	✓	-	3415S	3415M	3415L	-
bathing sling PVC	-	-	3464XS	3464S	3464M	3464L	3464XL
bathing sling PVC 6 strap	-	✓	-	3418S	3418M	3418L	-
Thorax hygiene sling	✓	-	-	3479S	3479M	3479L	-

SPECIAL	XS	S	M	L	XL
sling for stand-assist	-	3406S	3406M	3406L	-
walking harness	-	3471S	3471M	3471L	-
pool lift chair	-	-	3620	-	-
seat sling bath	-	-	3478	-	-
supine sling	-	-	3498	-	-
Low back sling	-	3491S	3491M	3491L	-
scoliosis sling	-	3499S	3499M	3499L	-
leg sling	-	-	3497	-	-
amputee - 1 leg	-	3494S	3494M	3494L	-
amputee - 2 legs mesh fabric	-	-	3540	-	-
amputee - 2 legs mesh fabric with hygiene cut out	-	-	3543	-	-
amputee - 2 legs solid fabric	-	-	3541	-	-
amputee - 2 legs solid fabric with hygiene cut out area	-	-	3544	-	-
amputee - 2 legs parachute nylon fabric	-	-	3542	-	-
amputee - 2 legs parachute nylon fabric with hygiene cut out	-	-	3545	-	-

Spreader Bars

For proper suspension in slings

If the SUREHANDS® body support is inappropriate or contraindicated, lifting can be done with one of the many SUREHANDS® slings. We offer innovative two and four point spreader bar systems. When combined with the multi-loop features on the SUREHANDS® slings, the caregiver has an opportunity to perfectly distribute the patient's body weight for maximum comfort and functionality.

Perfect weight distribution

On the four point spreader bar system, the patient can be positioned in a reclining, sitting or inclined position. The position is determined by using a combination of back and front loop attachments. For example, the longest length in the front and the shortest length in the back will create a sitting position.

The spreader bar is bow shaped so that the patient's head is out of the way during the initial lifting. Once the patient is lifted, this curved shape also prevents the patient's vision from being obstructed so the actions of the caregiver can be easily followed.


One mobile lift, two systems

Changing from the combi spreader bar to the SUREHANDS® body support is safe and easy.




Quickly and safely put on


The sling is quickly and safely put on and removed, saving time and energy for the caregiver. Also, there is no need for the patient to sit on the sling all day.

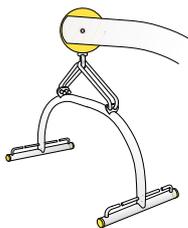

Combi spreader bar
uses hooks to attach to eye ring

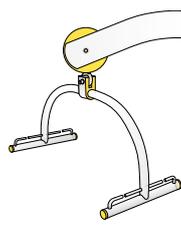

Standard spreader bar
attaches directly to eye ring

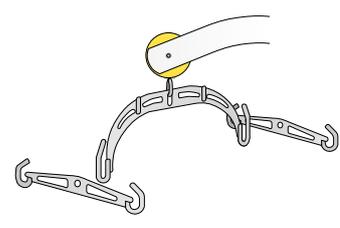

Classic spreader bar

HANDI-SPREADER BARS	easily interchangeable	treated/coated	stainless steel
Combi spreader bar (4 points)	•		•
Standard spreader bar (4 points)			•
Classic spreader bar (2 or 4 points)	•	•	





The 2600 series mobile lifts

Powerful, easy to use & durable

The 2600 series mobile lifts are designed for intensive use in a professional environment. The lifts are designed for efficient and frequent lifting of persons with limited mobility in facilities for persons with disabilities, such as residential care, hospitals, and other professional settings.

Easy to operate

The lift can be operated by using either the hand control or the buttons on the control panel. A mechanical emergency lowering mechanism ensures that the user can always be lowered in an emergency situation.

Maneuverable

Both robust and compact. The ergonomic handles make it easy to steer, even in confined spaces. The base features electrically adjustable width to fit perfectly around a wheelchair.

Battery life

The replaceable battery is charged directly on the lift or by using a wall-mounted charger. It has a long operating life: 160 transfers with a weight of 198 lbs.

Durable construction

The lifts are made from aluminum and stainless steel. This makes it easy to maintain and protects them against corrosion, which ensures a long service life. The base is finished with rubber bumpers. The durable wheels feature aluminum interior mechanisms and have a load capacity of 275 lbs each. They can easily be disassembled for cleaning.



Mobile lift model 2600/2610 (Victor)

Fast, powerful, versatile

The SUREHANDS Victor is a mobile lift for intensive use in a professional environment.



The fastest lift on the market

Thanks to its innovative drive system, the Victor is twice as fast as the average lift, even under maximum load.

Easy lifting up to 500 lbs

The Victor has a large lifting range, extending down to floor level. The user has plenty of leg room and can thus be turned around freely

Versatile

The Victor can be used together with the unique SUREHANDS Body Support® and slings. They can be interchanged easily to suit the specific needs of the user.



The Victor can be fitted with a digital weighing scale as an option



Stand Assist model 2620

Comfortable from sitting to standing

The Stand Assist is ideal for transfers from a wheelchair to the toilet, bathroom, bed or recliner. With the SUREHANDS Stand Assist, the lower body remains completely free. The caregiver can dress the user or change the incontinence material with little effort. This makes it ideal for visiting the toilet.

Little pressure under the armpits

The patented design makes the sit-to-stand movement comfortable, even for those whose weaker muscle tone prevents them from cooperating actively.

The central position of the attachment points and the sling design ensures it fits snugly around the torso and does not slide upward during the movement. As a result, the pressure under the armpits is minimal.



Natural 'stand-up' movement

The patented “combined movement of the lifting arm and handle”, provides an ergonomically correct sit to stand motion. The handle and the lifting arm follow an independent curve bringing the user to an upright posture via a natural “stand-up” movement.

Easily pass through the doorway

Hands and arms remain in a natural position next to the body to safely pass through doorways

Stable standing position

The natural movement, the ergonomic position of the hands, and the well-supported lifting sling provide a secure feeling and a stable standing posture. The knees rest against a soft knee support and can be fixed with an adjustable calf strap. The large lifting range of the lifting arm ensures that even taller people can still achieve a perfect standing position.

Due to the stable standing position, this active lift can also be used to support or maintain the standing function. The supportive design can actively lift people with less strength, delaying the need to switch to passive lifting. One lift accommodates a broader group of people.

Compact and maneuverable, suitable for small spaces

The small footprint, central center of gravity and low rolling resistance of the wheels make this an easily maneuverable mobile patient lift. Suitable for small spaces, such as the toilet.

The leg spread of the undercarriage is electrically controlled and occurs simultaneously with the rising movement for a quicker transfer. The wide range of leg movement makes this lift easy to use with wheelchairs and recliners.



The 1600 series mobile lifts

Quick and easy solutions for use anywhere

The 1600 Series mobile lifts allow users with limited mobility to be comfortably lifted and safely moved. These sturdy lifts are ideal for private use.

Maneuverable

Large wheels ensure the effortless maneuverability on any surface. They have a low rolling resistance, are maintenance free and the wheelbase is width adjustable. The lifts easily move through doorways.

Sturdy and safe

The SUREHANDS mobile lifts are durable and well-built. Constructed of high-quality materials and assembled by our in-house experts to meet the most stringent regulations, resulting in a longer service life. Equipped with higher capacity batteries these mobile lifts are able to handle up to 120 lift cycles between charges depending on the model.

Practical

Use either the magnetic remote hand control or the console switch on the control box to raise or lower the lift. The lifts can be easily disassembled for transport or storage. The low profile chassis makes it possible to slide the wheelbase under almost any bed.

Lifting from the floor

These mobile lifts are designed to lift a person from the floor.



Model 1640

This lift has an extended lifting range of 82 inches and is able to lift from the floor to a very high bed. With a weight capacity of 386 lbs, it is the most adaptable allowing for the use of the SUREHANDS Body Support®, SUREHANDS slings or for the addition of a weighing scale. One of the advantages of this larger lift is that it offers extra leg room for the user.



Model 1641

This lift has an integrated spreader bar to be used with SUREHANDS slings. It has a high lifting range to reach above a tall bed, raised bath or treatment table. The advantage of this lift, with a weight capacity of 440 lbs, is that it offers extra leg room for the user.



Model 1630

This compact lift is suitable for use in nearly every environment. With a weight capacity of 386 lbs, this lift offers the highest lifting range amongst its peers. Its shorter wheelbase and lower height make it ideal for use in tighter spaces. With an integrated spreader bar, the lift can be used with any SUREHANDS sling.



Model 1635 Foldable mobile lift

This lift is a compact, foldable unit engineered for daily use. It uniquely folds without the need for disassembly, thereby reducing storage space requirements. Its smaller footprint facilitates easier transportation to secondary locations, making it an ideal option for a second home or visits to grandparents. The lift features a built-in charger and an exchangeable battery box for continuous operation. The legs of the lift can be spread using a foot pedal, enhancing ease of use. With a weight capacity of 386 lbs., this lift incorporates an integrated spreader bar compatible with any SUREHANDS sling, ensuring versatility and reliability in handling.

Model 1620 Stand Assist

Quickly to and from the toilet

Users who, because of reduced muscle tone, have difficulty in getting up from a sitting position can quickly and easily go to the toilet thanks to the Stand Assist.

The lower body remains free

The big advantage of the Stand Assist is that the lower body remains free so that lower body clothing can be removed without difficulty. With the help of a sling supporting the back and under the arms the user is lifted into a standing position.

The sling is fitted quickly and easily without placing too much pressure on the shoulders. The system is designed to support the body as much as possible, without overloading the shoulder line.

The Stand Assist is also an excellent aid when dealing with incontinent patients.

Support and balance

The user's feet are placed onto the footboard of the Active lift while the user is in a seated position.

During lifting and moving the user's lower legs are supported by the adjustable knee pad. The knee pad helps to keep the body in balance during lifting and standing.

Compact and maneuverable in small rooms

The Stand Assist is easy to maneuver in tight spaces because of the short base length. The base is also adjustable in width to go around wheel-

chairs or geriatric chairs. It is precisely because of its compact design and features that the Stand Assist is ideal for transfers in small rooms such as bathrooms and toilets.

Removable and exchangeable battery holder with charger

The Stand Assist is operated by a battery located in an exchangeable battery holder incorporated into the device. The battery section of this assembly is removable from the charger so that it can be easily exchanged with a freshly charged battery. Or, it can remain on the unit for charging when not in use. An LCD screen indicates when the battery requires charging. A fully charged battery can carry out 40 lifts.

For those who use the Stand Assist continuously a removable battery pack is ideal: while you are using one battery the other is being charged.



2800 Series Ceiling Lift

Continuous movement without obstacles

Imagine: you want to move freely in all spaces without being hindered by doorsteps, cupboards, carpets or obstacles on the floor. Realize your dream withan 2800 series ceiling lift with fixed motor for continuous movement from room to room.


Ideal for independent use

If you have lost the ability to transfer manually but you still have a limited range of motion with adequate muscle tone, it may be possible for you to self-transfer with either the 2800 L or 2800 LRC track system. Go from your bed to wheelchair, toilet or bath in silence with a soft stop and start motion. The LRC wireless infrared remote model gives you the ultimate control to send the motor away when not needed and summon it back when you are ready to go.



Efficient and quick solution for Attendant care

Let the motor help you with the strain of lifting and lowering. Easily transport the person laterally along the track system once they are suspended. Transfers to bed, bath or toilet have never been easier. All SUREHANDS track systems can be used with either the SUREHANDS Body Support® or SUREHANDS slings. So everyone can be accommodated regardless of their disability. SUREHANDS® track systems are perfect for professional as well as private applications. The 2800 series motor works with an electric handcontrol; the 2805 IR works with an infra-red handcontrol.

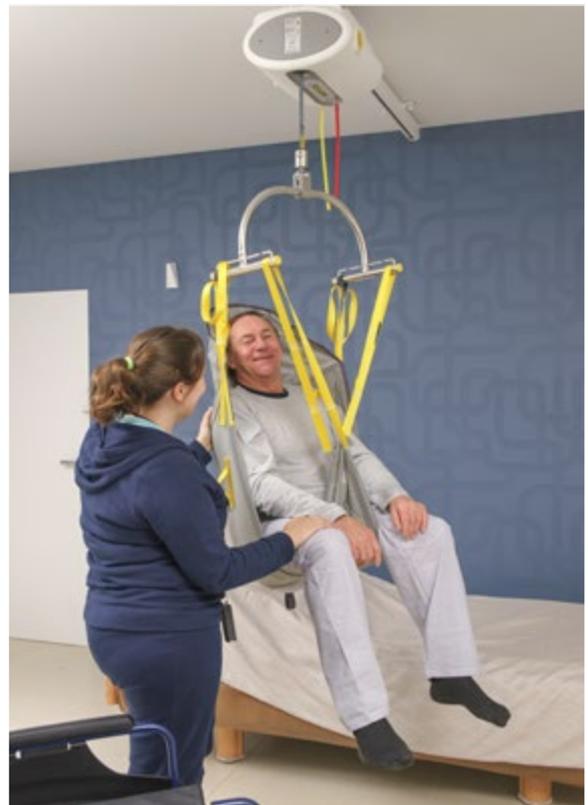

Customized solutions

Avoid costly remodeling of spaces that are not ideally set up for transfers. SUREHANDS offers numerous options from which to choose, e.g. curves, traverse track, switches and turntables, just to name a few.

Discreet appearance blends with each interior

With its white finish and muted, aesthetic styling, the ceiling motor blends nicely into most interiors. The motor, which operates on rechargeable batteries, has a lifting capacity up to 440 lbs and an attendant care version up to 665 lbs.

Safe use

As an additional safety measure, dual controls on the motor and a mechanical emergency down has been incorporated that can be operated independently.

Distance control anywhere on the ceiling track

SUREHANDS provides magnetic contacts for use with the 2800 L and 2800 LRC ceiling motors, which can be placed anywhere along the track to insure that the motor will stop at certain predetermined places, e.g. middle of the toilet, on the charging pins, etc.



2840 Series Portable Ceiling Lift

A single motor for multiple locations

SUREHANDS® Portable Ceiling Lift can go track-to-track offering a flexible solution to lift and transfer in different rooms that are not connected by a continuous rail system. Move the Portable Ceiling Lift between rooms, floors, or different homes.

Use in different rooms

SUREHANDS® installs the rail where required and the Portable Ceiling Lift can be moved. All rooms are accessible, regardless of the size or layout. Wherever rails are provided, the Portable Ceiling Lift can be attached, facilitating transfers at the bed, toilet, tub or even the garage. Even staying at the vacation home or at the grandparents is no longer a problem: taking the Portable Ceiling Lift along enables the grandparents to easily lift and look after their grandchild. Small rooms offer little in terms of maneuverability. A combination of a rail system and a Portable Ceiling Lift provides a clever solution for places that are difficult to access. For example, the bathroom may no longer need to be modified. The tub can stay.

Light, compact and aesthetic

The user-friendly Portable Ceiling Lift is no bigger than a shoe box and weighs less than 17.5 lbs. so that it can easily be carried along by its handle. Using the reacher, the motorband can quickly and effortlessly be hooked on and off the rail, without the need for a ladder or chair. The Portable Ceiling Lift can be stored out of sight when not in use.



No changes to your home

To use the Portable Ceiling Lift, no changes need to be made to your home. Everything remains in place. Door openings do not have to be modified as the system works with individual rail sections maintaining the integrity of your living space.

Budget friendly and multi-functional

Using a single Portable Ceiling Lift for multiple locations offers a budget-friendly option. When paired with a sling or the Body Support, the Portable Ceiling Lift meets all user requirements and needs. Capable of effortlessly lifting 400 lbs., the Portable Ceiling Lift features long-life rechargeable batteries that can be conveniently charged using any standard socket with a separate charger. Furthermore, power in or around the rail is not necessary for operation. The Portable Ceiling Lift can be controlled manually or via the control panel on the motor itself and is supplied with an emergency down feature.

"Tarzan" feature

With the "Tarzan" feature, users can move from one room to another without a wheelchair and without the need to remove door headers, providing added convenience



Ceiling Track Installations

Effortless transfers from one place to another

Whether it is the 2800 series Ceiling Lift or the 2840 series Portable Ceiling Lift, SUREHANDS® provides multiple ceiling track possibilities perfect for the private home or professional environments and facilities.

Aesthetically designed with quiet operation

SUREHANDS® durable Ceiling tracks are constructed of a strong aluminum alloy and enameled in high gloss white to blend with most ceiling colors. The multiple and small synthetic wheels within each motor system ensure that the sound is kept to a minimum as the motor travels along laterally.

Customized solutions

SUREHANDS® specialists work with designs for new construction as well as existing spaces. Practicality and simplicity are blended with functionality to achieve the best solution for each client. Scale drawings are provided to architects, designers, builders and homeowners to ensure that the solution will work before it is installed.

Tracks can be attached to the ceiling or hung from brackets mounted to each wall.



Ceiling suspension

The safety of your installation is increased by requiring ceiling support brackets every 48 inch / 120 cm. In addition, the two bolt support bracket design further increases the safety redundancy of your installation. Straight track or 30, 45, 60 and 90 degree curves are available. Even mounting through dropped ceilings is possible. Additional switches and turntables make it possible to switch tracks for even greater accessibility.

Reinforced track for wall-to-wall mounting



In installations requiring wall to wall mounted rail, your safety is again increased by the manufacturer's reinforcement of the rail up to 23 ft / 7 m when hung only from a wall bracket on each wall. In situations where the ceiling is not easily accessible or not strong enough e.g. cathedral or sloped ceilings, the reinforced rail can be a good option.




Points / Turntable

Change Direction to access 3 or 4 areas


The electric points and turntable allow for a ceiling motor to travel in different directions and rooms. Maybe you want to go from a bedroom off the main hallway to the tub or shower in a corner of the bathroom and the toilet that is not in line. Both areas can be reached with the push of a button changing the direction of the track just like on the railroad. And with the 2800 LRC motor and the RC version of the electric points, this can all be accomplished using just one wireless infrared hand control. Even in small areas multiple directions can be accessed



Points

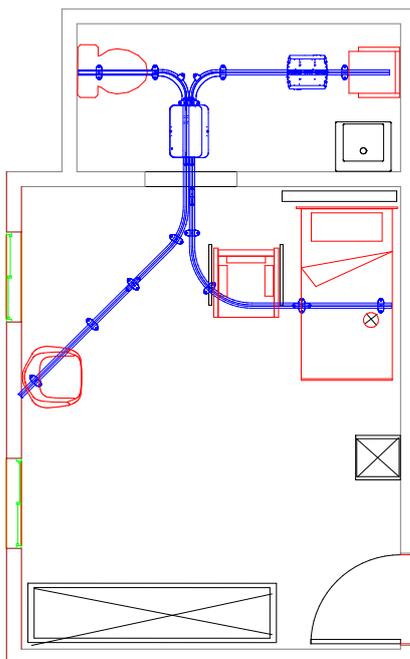

Turntable

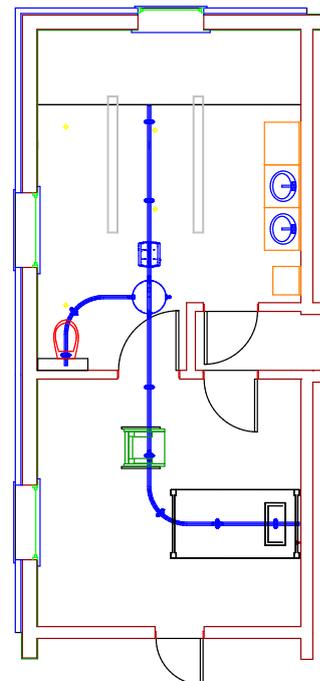

Always the right solution

These multiple ceiling track options make it possible for everyone to meet their transferring needs with a SureHands® system. Our specially trained representatives will help you find the correct combination.

Traverse Track Systems

Covering an entire space

When it is necessary to lift from any point in a room, the SUREHANDS® traverse track system is ideal. This system consists of two fixed ceiling or wall mounted parallel rails and one fortified traverse rail that moves back and forth on the parallel rails. This is ideal for large therapy spaces or living areas with multiple wheelchair users or where partitions that don't reach the ceiling divide spaces but still allow the traverse rail to move overhead. Total room coverage is achieved from one Ceiling track system.

With or without assistance

When self-lifting and transferring is the goal and total room coverage is necessary, our motorized version of the traverse track system is the ideal solution. And again, the motor and the traverse

can all be controlled from the same wireless infrared remote control.

Traverse connector

A traverse connector allows the traverse rail in one room to connect to a single rail in the next room. In one smooth movement transfers can be made in multiple areas.





The Freedom Bridge

A flexible solution

The freedom bridge is an ideal solution when installation on the ceiling or walls is not possible, or when the situation is only temporary.

This system is easy to set up and relocate. No holes need to be drilled in any walls or the ceiling.

The Freedom Bridge is compatible with both fixed and portable ceiling track motors.



The Wall Lift System

Easy installation

The SUREHANDS® Wall Lift System is easy to install and can be used near the bed, toilet, bath, shower etc. The portable lifting device can be mounted on the wall or a post.

Multiple mounting options

The wall lift system is easy to install. Existing homes may have technical limitations that require modifications for certain lift solutions. This is not the case for the wall lift, as it can literally be mounted anywhere. If the wall or partition is not strong enough, SUREHANDS® will install a post.

Adequate lifting range

The wall lift has a 180° rotating range, an extendible boom arm and an adequate lifting range. The extension further increases flexibility and the lifting range.





Economical and multi-functional

As the wall lift system can be used in different places, it offers an economical solution. One boom arm can be moved to multiple areas, for example, from the bathroom to bedroom. The wall lift could be installed between the bath and toilet, reaching both areas to transfer. Then moved to the bedroom to transfer to the bed. It can be used with the spreader bar and sling or a Body Support.

No modifications to your home

Is your bathtub not accessible with a mobile lift? Is your ceiling too low or not strong enough for a rail system? The wall lift is installed to the wall or a pole, so no modifications to your home are required. Everything remains as it is.

Safe and reliable

The wall lift system has a built-in safety mechanism. The emergency down feature enables the boom arm to be always lowered. In addition, the manual control is waterproof. The wall lift operates from AC mains and has an emergency stop.



The Motorized Wheelchair-to-Water™ Pool Lift

Go directly from the wheelchair into the water with the innovative SUREHANDS® motorized Wheelchair-to-Water™ pool lift. Raise the user from the wheelchair with the power of the lifting motor and manually rotate the arm effortlessly until the user is over the water. Lower with the same powerful motor. Remove when not in use to store or recharge the batteries.

User friendly

The SUREHANDS® Wheelchair-to-Water™ pool lift is compact, ergonomically designed and easily transportable on its four wheels.

Quick to set up

Roll the motorized Wheelchair-to-Water™ pool lift to one of the sockets at the edge of the swimming pool. Center over the socket and press the button to engage and raise the arm to the desired height. Attach the desired SUREHANDS® accessory and lift the user. Manually rotate the arm over the swimming pool and lower the user



into the water with the press of a button. The lifting arm is perfectly flexible with a 360 degree rotation.

Built to last in wet conditions

The Wheelchair-to-Water™ pool lift is made of sturdy stainless steel, operates on batteries and has a waterproof hand control for safer use around water.




Great mobility

Because it is easy to engage and disengage from the socket and easy to roll away when not in use, the lift never needs to block the perimeter of the pool when not in use.



Other SureHands® options for lifting in the pool

The 2800 series Ceiling Lift offer unique access to the pool. Ideal when multiple transfers are needed or where ambulation or other therapeutic applications are desirable. Also ideally suited for indoor swimming pools with somewhat limited deck space.

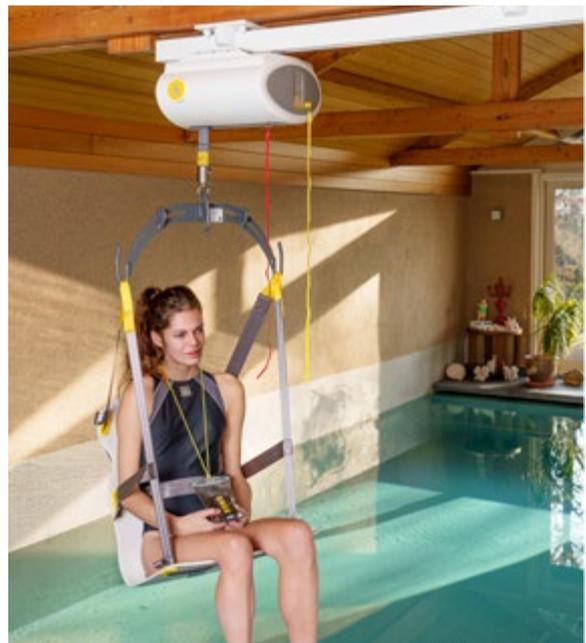

The shower-bathing trolley

3-in-1 system: shower, bath and treatment

This unique, patented, and height-adjustable shower bathing trolley allows the caregiver to efficiently and ergonomically shower or bathe a patient while always being at the patient's side. The shower trolley can also be used as a treatment table.

Multi functional

The space saving SUREHANDS® shower-bathing trolley combines 3 functions: bath, shower stretcher and treatment table. The shower trolley is mobile, accessible on both sides and accessible underneath with a mobile lift.

With the use of the roll board, a patient is easily moved from the bed to the shower trolley and back.

High/low

The double electrical high-low functionality enables the caregiver to lift the patient to the correct working height and to raise or lower the sides of the shower trolley to create a shallow bathtub or flat shower stretcher. The shower trolley can be raised very high, ideal for tall caregivers, and very low to allow direct wheelchair transfers.

With the sides in their highest position, it serves as a shallow bathtub.





Comfortable and easy to maintain

The shower trolley is pleasantly warm, soft and comfortable. The synthetic fabrics are easily maintained and because the sides are rounded with no seams they can be hygienically cleaned. A waterproof pillow comes standard with the shower trolley.

Designed with safety in mind

The shower trolley is operated with a waterproof electrical manual control with spiral cable. The exchangeable battery box (24v) is charged separately in a wall charger.

Removable side bars

If possible, the patient is able to actively help during bathing or treatment by turning sideways and simultaneously pulling up to one of the side bars. The side bars can also be used for additional safety during use or transport.

Durable

Construction is made from corrosion resistant materials. A combination of RVS, aluminum, powder coated steel and durable synthetics.



Transport Trolley for the Portable Motor

An ergonomic solution for the caregiver

The portable motor weighs only 17.5 lbs, and is easily moved from track to track. Caregivers who prefer not to carry the motor can use the trolley to transport both the portable motor and the Body Support at one time. Move from room to room with ease while reducing the risk of dropping the motor.

This compact trolley has a hook at an ergonomic working height, making it easy to unhook the portable motor and SUREHANDS® body support from the rail system and quickly hang them on the hook of the transport trolley to move them to another room.

It is a compact device that does not take up much space, it is easy to maintain, and hard-wearing due to stainless steel construction.



Scales

Weighing and lifting simultaneously

Monitoring weight is made simple and accurate. Maintaining a healthy weight can be essential to a care plan and can improve quality of life. The digital scale is designed to work in combination with the SUREHANDS lifting systems. Calibrated and certified to an accuracy of 0.1%, verifying weight is easy and precise.

Designed for SureHands

Designed to be used with most SUREHANDS lift systems. It is suspended from the swivel hook of the lift system and connected below to the SUREHANDS® Body Support or Spreader Bar. The scale is easily attached and detached and is secured with a safety retainer screw during weighing.

Compact

The scale is compact and very light, making it extremely easy to handle.



Accurate

The scale is provided with a Certificate of Calibration to an accuracy of 0.1%. It is reliable for people who need their weight monitored for medical reasons. The weight is displayed digitally on an LCD screen.

Easy to use

The scale has three buttons: “lbs/kg” button allows you to select between lbs/kg on the display. The “weigh” button is pressed to display the weight data. The “zero” button is used to zero out the scale before lifting. When the patient is lifted, the weight of the scale is deducted automatically and the scale will record their actual weight.



A powerful formula

Friendly and professional advice

We believe in the principle that a good customer relationship begins with good information. Our representatives are pleased to put all their knowledge at your disposal, when you are ready, without any obligation or charge.

Clear agreements

SUREHANDS® wants your satisfaction before, during and after your purchase. Therefore, our representative will provide you with a clear-cut picture of what you have ordered and how the delivery and installation will be carried out.

Customized solutions

Your SUREHANDS® representative works with you to select the best system for your budget and situation. Behind your representative is a team of customer service and technical personnel to offer additional support. Your request is treated uniquely and professionally.

Solid warranties

Each SUREHANDS® system is manufactured to rigid standards and specifications. But should something go wrong, you will be pleased to know that SUREHANDS® offers some of the best warranties in the industry.

Complete service after purchase

SUREHANDS® and its representatives will remain your loyal partner after the purchase as well. You will receive a comprehensive instruction guide and personal training on the usage of your system. A toll-free customer service number is available afterwards for any concerns or for additional information.





SUREHANDS LIFT & CARE[®]

982 County Route 1
Pine Island, NY 10969
Tel:(800) 724-5305
Fax:(845) 258-6634
E-mail: info@surehands.com
www.surehands.com

